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Name of the medical device: Colorodent

The medical devices "COLORODENT BASIC, COLORODENT GOLD and COLORODENT SUPERCOLORI" can be used to infiltrate substructures and/or anatomical elements (FULL CROWN) in a homogeneous way, before sintering. There are 16 colours available in addition to the VITA® scale. COLORODENT infiltration paints are not acid-based and do not require stabilizing liquids.

IT IS NOT POSSIBLE TO MIX THE "COLORODENT" WITH OTHER DYE LIQUIDS HAVING THE SAME REFERENCE TO THE LIFE SCALE OF OTHER MANUFACTURERS. IT IS IMPORTANT NOT TO USE HEATING LAMPS OR HAIR DRYERS DURING DRYING, AS THEY COULD OXIDIZE THE LIQUIDS WITH A SIGNIFICANT VARIATION IN THE COLOR RESULT. SHAKE THE PACKAGING BEFORE PROCEEDING WITH USE. Do not use metal tweezers that may stain the frame. The structures to be infiltrated must be clean and dry. Store liquids out of direct sunlight.

Zirconia ORODENT dry milled:

Recommended use:

Clean the frames from dust using a brush and air jet. (N.B. Do not use toothbrushes or abrasive tools that may deteriorate the integrity of the structures).

Completely immerse the pre-sintered zirconia frame in the dye liquid.

Immersion time depending on the type of material used. (see table at the bottom of the page)*.

Remove the frame from the dye and place it on paper towels for a few minutes.

Remove the excess liquid that forms inside the structures with a not too strong jet of air oriented towards the frame at a distance of about 20cm.

Proceed with the sintering of the pieces.

*For the staining of the various types of zirconia, use the following table of the corresponding liquids:

TYPE LIQUIDS	ZIRCONIA	INFILTRATION TIMES
COLORODENT Basic	White Matt, High Translucent	15"
	Bleach	10-15"
COLORODENT Gold	Thor, Eos, Venus, Preshaded and Gold	20"

For the choice of processing methods and type of materials, consult the Zirconia LAVCAMZR user manual available online on the www.ordent.com website or by requesting it directly at 045/6450635*.

 **WARNINGS AND CONTRAINDICATIONS**

Handle the product after consulting all the other sections of this sheet. Do not eat, drink or smoke during use.

Ensure good ventilation in the workplace by means of effective local extraction or exhaust of stale air.

-HAND PROTECTION: Protect your hands with category I work gloves (ref. Directive 89/686/EEC and EN 374) such as latex, PVC or equivalent. Wash with soap and water after removing protective clothing. Storage and shelf life: Store dye liquids in their original packaging in a clean, dry place.

Temperature 0-35°C.

Do not use past the Expiration Date: See label on bottle.

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