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## Name of the medical device: Orodent

Please read this instruction carefully before removing zirconia disc from the packaging. This instruction contains information for safe processing of the zirconia material. Please note processing can only be carried out by expert healthcare professionals (dental technicians) under written medical prescription from the dentist.

**Description:** Orodent Medical Devices are made of zirconium dioxide with partially stabilised Yttria tetragonal structure. They are available in different sizes, for milling using dental CAD/CAM equipment. The intended use for design and manufacture of dental prosthetic structures can be found in the technical sheets of each product, classified according to EN ISO 6872 (E), table 1.

### Recommendations for use:

**Processing:** Orodent Medical Devices can be milled dry or wet using distilled water. In the event of dry milling, this material is forming a very fine powder, which can cause irritation to eyes and skin and can damage the lungs. Therefore, please make sure that the necessary exhaust system is operating perfectly. In case of processing using cooling lubricants, only use biocompatible products.

It is advised to adequately clean the prosthetic structure after milling to eliminate any leftover milling residues and dust.

During the sintering phase (in the sintering oven at 1530°C) the milled structure undergoes approx. 20% shrinkage. The use of appropriate supports that sustain the structure without impeding their movement is recommended. Place structures in the oven keeping them equidistant from the heat source irradiation.

For the choice of possible working and processing methods as well as types of materials, please consult the LAVCAMZR Zirconia User Manual available online at [www.oro dent.com](http://www.oro dent.com) or by requesting it directly at +39045/6450635.

**Finishing:** After sintering process, the structures only with water-cooled tools which are in perfect working condition. Avoid overheating of the ceramic body as doing so may cause damage to the structure.

**Ceramisation:** use zirconia specific ceramics (according to EN ISO 6872) with a heating gradient of 20 to 35°C depending on the thickness of the restoration.

**Storage:** The material can be stored under normal conditions (in a dry and ventilated place) without any special precautions.

**Disposal:** avoid release into surface water, ground water or ditches. Dispose processing residues according to current legislation, through authorised disposal companies, as non-hazardous waste (CER 120104).

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## **WARNINGS AND CONTRAINDICATIONS**

To guarantee traceability, it is advisable to keep the device in its original packaging with the instructions until complete use.

The medical device has low residual risks. The following indications refer to the powders that are released during the processing of the medical device.

During processing: vacuum the dust from the object with appropriate equipment; do not inhale the powder; keep food and beverages away from the workplace; do not smoke, eat or drink while working. After working, clean your hands and treat the skin.

To protect the operator: wear gloves for the hands and wear protective glasses for the eyes.

In case of contact with powder, wash thoroughly with water and consult a doctor In case of ingestion: rinse the mouth and consult a doctor Protect the mouth with a dust mask and wear light protective clothing.

In relation to the patient, the use of the ORODENT medical device is contraindicated in the case of patients with untreated bruxism, temporary cementation and in all other uses not listed in the indications\*. The actual and general assessment of the suitability of the type of medical device to be used for a specific patient remains the responsibility of the dentist.

The processing, handling and application of the product carried out in the dental clinic or laboratory are outside the control of the manufacturer and therefore fall under the responsibility of the user.

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